

VIVEREX®

Your simple single use
Negative Pressure
Wound Therapy system



Everything you need to know

Dear Patient,

We understand how important effective wound treatment is and to make sure you get the most successful therapy outcome with the VivereX® we have created this booklet.

Here you will find answers to all the most frequently asked questions regarding the use of the VivereX®, in addition to helpful tips for its application that can assist in promoting proper wound healing.

Please remember however, that only healthcare professionals with the proper qualifications should apply wound dressings.

We wish you a speedy recovery and are here to support you every step of the way!

Tell me more.

1. How does the VivereX® wound treatment system work?
2. A quick guide to the VivereX® device and its parts.
3. How can I incorporate the VivereX® therapy into my daily routine?
4. A step by step guide to showering with the VivereX?
5. What do each of the indicator lights mean and what should I do?
6. When should I get in touch with my healthcare provider regarding the VivereX® treatment?




How does it work?


The VivereX® system consists of a lightweight pump that is connected to an adhesive dressing, which is applied to the wound. The pump is pre-programmed to apply a specific amount of negative pressure to the wound surface

The VivereX® device uses this negative pressure to effectively drain wound secretions (exudate) from your wound, which helps to promote faster healing. In addition, it may improve blood flow to the wound, supporting the formation of new, healthy tissue.

The system is designed to work in harmony with your body's natural healing processes, helping to speed up the healing process and prevent further complications.

Quick guide

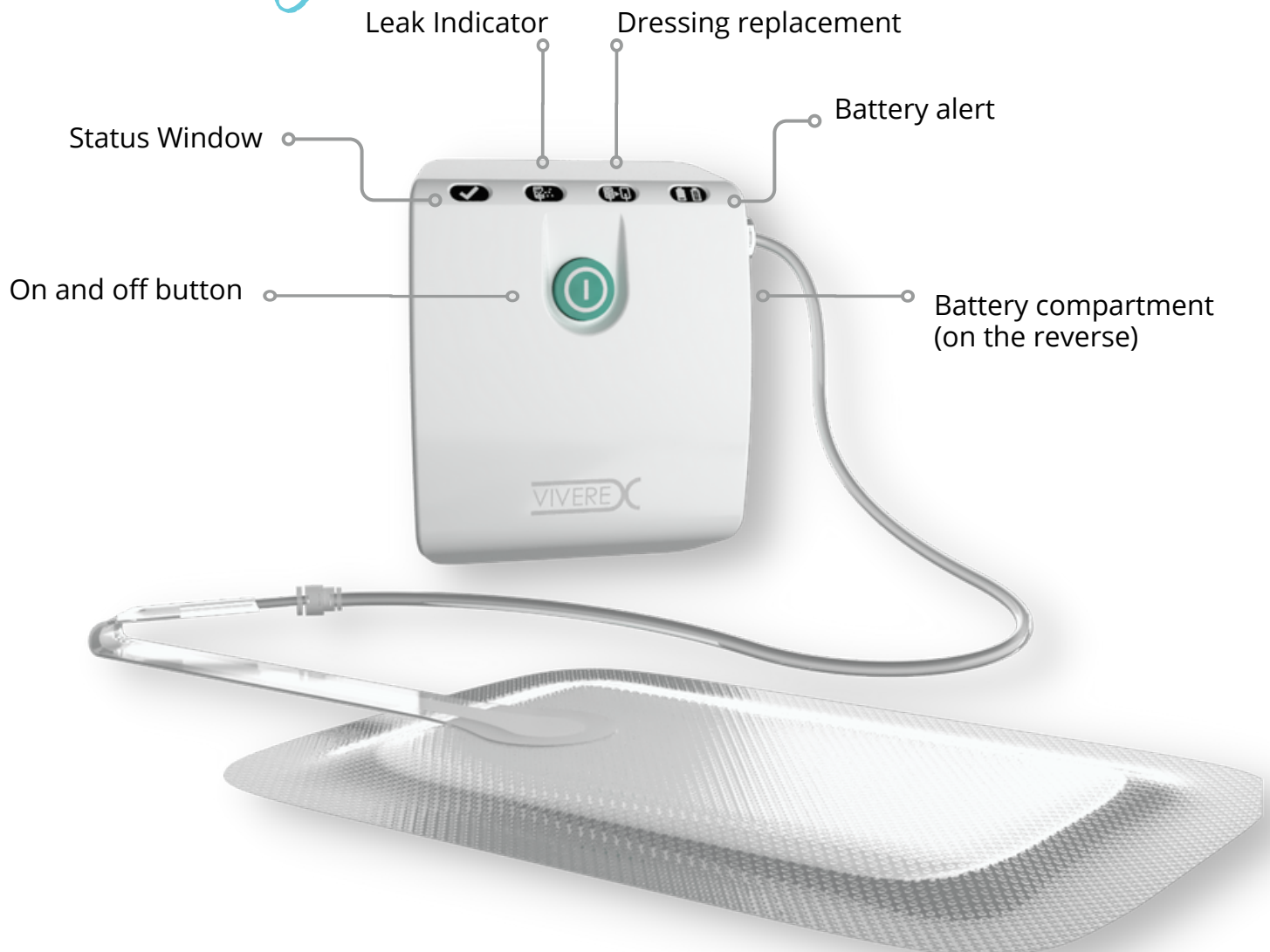

1. Your health care provider will dress the wound appropriately using one of the supplied Viverex® dressings
2. Once the wound is sealed the pump will be turned on with the **On and Off** button.
3. The device is functioning properly when you can see this  in the **Status Window**.
4. The pump is equipped with alerts to show you when to change dressing, advise if there is a leak or if the battery needs replacing (view page 12 for more information on alerts)



Out and about

Provided that the dressing remains secure and connected to the VivereX® pump, **you should be able to move around without restrictions** while receiving wound therapy.

You can place the device into a pocket in your trousers or use the included belt clip - the design of the VivereX® pump allows for discreet and comfortable wearing.

Sleeping

It is important that you do not switch off the therapy device at night. Position the VivereX® device in such a way that it cannot fall down during the night. Ensure that the tubes do not become kinked.

Showering

You must disconnect the VivereX® pump before showering.

Step by step guide

- 1 Press the power button for three seconds to put the device in 'Off' mode.
- 2 Disconnect the VivereX® and keep the device in a safe place.
- 3 The end of the dressing's tube should be facing down, to prevent water from entering the tube.
- 4 The dressing should not be directly submerged in water or water jets.
- 5 After showering, reconnect the dressing to the device.
- 6 Press the power button to put the device in 'On' mode.
- 7 Ensure that the green wound therapy indicator is flashing, the dressing is connected and no indicators are active



Cleaning

You can clean the surface of the VivereX® device if needed with a soft cloth slightly moistened with water.

Note

Please consult with your healthcare provider or therapist for any specific instructions on movement limitations or device usage during your therapy, as individual requirements may vary.



Indicators

The four indicators at the top of your device operate as follows:



When this status window is green and flashing, it indicates the device is functioning normally



If this amber leak indicator is flashing it indicates that the dressing is NOT airtight




If the dressing indicator is flashing amber, it indicates that the dressing needs to be changed. Please contact your healthcare provider.



If the battery indicator is flashing the power is too low to operate the device and the batteries require **changing within 2 or hours.**



If the BOTH the green status window AND the battery indicator turn on this indicates the device batteries are running low and should be **replaced within 24hrs**

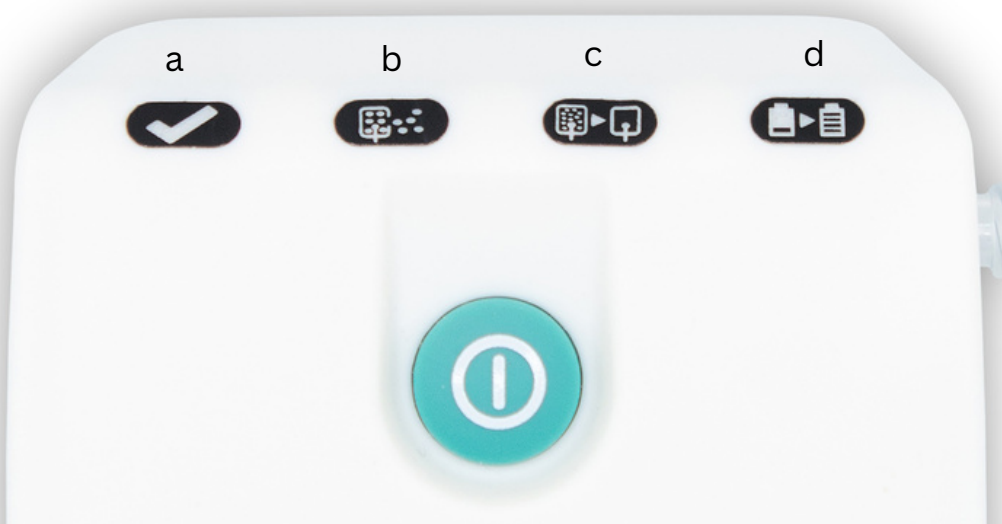

When to contact your healthcare provider?

- 1 If the dressing comes off.** Cover the wound with a sterile dressing.
- 2 If any of the indicator lights are flashing amber.** You may need the dressing or battery changed.
- 3 If you notice the pump is making any noises that sound concerning.**
- 4 If you have any signs of infection.** For example swelling, redness around the wound, pain, pus or fever.

